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PRINCIPLE INTENDED USE

The On Call[®] Plus Blood Glucose Test Strips are thin strips with a chemical reagent system. They work with the On Call[®] Plus or On Call[®] EZ II Blood Glucose Meters to measure the glucose concentration in whole blood. Blood is applied to the end tip of the test strip, then automatically absorbed into the reaction cell where the reaction takes place. A transient electrical current is formed during the reaction and the blood glucose concentration is calculated based on the electrical current detected by the meter, then the result is shown on the meter display. The meters are calibrated to display plasma-like concentration results. For *in vitro* diagnostic use. Test strips are to be used only outside the body for testing purposes. For self-testing and professional use. On Call[®] Plus Test Strips are used by people with diabetes at home and by healthcare professionals for the quantitative measurement of glucose in capillary whole blood from the finger, forearm, and palm.

COMPOSITION

Each test strip contains the following reactive chemicals: Glucose oxidase < 25 IU, Mediator < 300 µg. Each test strip vial contains a drying agent.

STORAGE AND HANDLING

- Test strips should be stored in their protective vial. The vial's cap must be tightly closed. This is to keep the test strips in good working condition.
- Store test strips in a cool, dry place at room temperature, 2-35°C (36-95°F). Store them away from heat and direct sunlight.
- Do not freeze or refrigerate.
- To ensure accurate results, use the test strips at room temperature.
- Do not store the test strips outside their protective vial. Test strips must be stored in the original vial with the cap tightly closed.
- Do not store or use the test strips in a humid place such as a bathroom.
- Do not store the meter, the test strips or control solution near bleach or cleaners that contain bleach.
- Do not transfer the test strips to a new vial or any other container.
- Replace the vial cap immediately after removing a test strip.
- Use the test strip immediately after removing it from the vial.
- Do not use your test strips past the unopened expiration date printed on the vial. Using test strips past the expiration date may produce incorrect test results.
- Note:** All expiration dates are printed in Year-Month-Day format.
- Use a new vial of test strips for only 6 months after opening. The opened vial expiration date is 6 months after the vial was first opened. Write the opened vial expiration date on the vial label after you open it.

PRECAUTIONS

- For *in vitro* diagnostic use. The test strips are to be used only outside the body for testing purposes.
- Do not use test strips after the expiration date shown on the vial. Expired test strips may give incorrect blood glucose readings.
- Do not use test strips that are torn, bent, or damaged in any way. Do not reuse test strips.
- The sample must only be applied to the tip of the test strip. Do not apply blood or control solution to the top of the test strip as this may result in an inaccurate reading.
- Before running a blood glucose test, make sure that the code chip contained with that vial of strips is inserted into the code chip slot on the right side of the meter.
- Discard the vial and any unused test strips 6 months after you first open it. Constant exposure to air may destroy chemicals in the test strip. This damage can cause incorrect readings.
- Keep the test strip vial away from children and animals.
- Consult your physician or healthcare professional before making any changes in your treatment plan based on your blood glucose test results.
- These products are labeled with the  symbol to distinguish them from earlier product versions that were subject to the EN ISO 15197:2015 non-compliance. Test strips with the symbol are compatible only with the control solutions that have the same symbol. The symbol can be found on the boxes, labels and individual pouches.

MATERIALS PROVIDED

- Test Strips
- Code Chip
- Package Insert

MATERIALS REQUIRED BUT NOT PROVIDED

- Meter
- Sterile Lancets
- Lancing Device
- Control Solution

INSTRUCTIONS FOR USE

See your User's Manual for complete instructions for blood sample collection before use.

- Open the cap of the test strip vial only to remove a test strip for testing. Replace the cap immediately to protect the remaining test strips from moisture in the air.
- Run the blood glucose test following the instructions contained in your User's Manual.
- The blood glucose test result will be shown on the meter display window. This result should fall within the target range recommended by your healthcare professional. If your blood glucose test results are higher or lower, ask your healthcare professional what to do. Always consult your healthcare professional before making any changes to your treatment plan.

IMPORTANT: On Call[®] Plus and On Call[®] EZ II Blood Glucose Monitoring Systems allow alternative site testing for forearm and palm testing in addition to fingertip testing. There are important differences between forearm, palm and fingertip samples that you should know. Important information about forearm and palm glucose testing:

- When blood levels are changing rapidly such as after a meal, insulin dose or exercise, blood from the fingertips may show these changes more rapidly than blood from other areas.
- Fingertips should be used if testing is within 2 hours of a meal, insulin dose or exercise and any time you feel glucose levels are changing rapidly.
- You should test with the fingertips anytime there is a concern for hypoglycemia or you suffer from hypoglycemia unawareness.

RANGE OF EXPECTED BLOOD GLUCOSE VALUES

Blood glucose monitoring requires the help of a person who knows their own range of expected blood glucose values and the meaning of your blood glucose results.

Expected blood glucose levels for people without diabetes:

Time	Range
Fasting and Before Meals	70 - 100 mg/dL
2 Hours After Meal	Less than 140 mg/dL

CHECKING THE METER

Your blood glucose meter must be handled according to the instructions for meter care. The quality control test strips are working together properly. Follow the instructions for a quality control test. Three ranges CTRL 0, CTRL 1, and CTRL 2. Control Solution 1 is sufficient for most meters. If your strips may not be working correctly, you may want to contact your dealer for information on purchasing control solution. For confirmation of results, Control Solution 0 test strips should fall within the CTRL 1 range. For results to be within the CTRL 2 range. When testing with Control Solution 1, the results to the CTRL 1 range on the vial label. **CAUTION:** If your quality control test result falls outside the CTRL 1 range, DO NOT use the system to test your blood glucose. If you cannot correct the problem, contact your dealer.

LIMITATIONS

- The On Call[®] Plus and On Call[®] EZ II meters are not to be used for the On Call[®] Plus and On Call[®] EZ II Blood Glucose Monitoring Systems. They have been tested and proven to work together effectively for blood glucose measurements. Do not use components from other meters.
- Use only with whole blood. Do not use with plasma or serum.
- Do not use for testing neonates.
- Very high (above 60%) and very low (below 20%) blood glucose levels may affect the accuracy of the results. Talk to your healthcare professional to find out more about these levels.
- Abnormally high levels of vitamin C and other substances may affect the accuracy of the blood glucose measurements.
- The system is tested to accurately read the range of 1.1-33.3 mmol/L (20 - 600 mg/dL). Fatty substances (triglycerides) up to 3,000 mg/dL may affect the accuracy of the results.
- The On Call[®] Plus and On Call[®] EZ II Blood Glucose Monitoring Systems are shown to work properly up to 10,000% (33.3 mmol/L) of blood glucose.
- Severely ill persons should not run the glucose Blood Glucose Monitoring System.
- Blood samples from patients in shock, or with hypotension, or with hyperosmolar state (with or without ketosis) may give inaccurate results when tested with On Call[®] Plus and On Call[®] EZ II Blood Glucose Monitoring Systems.
- Dispose of blood samples and materials carefully. Do not dispose of materials in the trash.

PERFORMANCE CHARACTERISTICS

The On Call[®] Plus and On Call[®] EZ II Blood Glucose Monitoring Systems meet the requirements of EN ISO 15197:2015 (*in vitro* diagnostic systems for self-testing in blood glucose monitoring systems for self-testing in blood glucose monitoring systems) and On Call[®] EZ II meters are calibrated by the Analyzer reference instrument, which is traceable to the International Union of Pure and Applied Chemistry (IUPAC) reference instrument.

Reproducibility, Precision

Ten replicate assays were each run on ten On Call[®] Plus and On Call[®] EZ II meters. The following reproducibility, precision estimates were obtained from the ten replicate assays.

MEAN	2.1 mmol/L (38 mg/dL)	4.9 mmol/L (89 mg/dL)
Standard Deviation mmol/L (mg/dL)	0.08 mmol/L (1.4 mg/dL)	0.14 mmol/L (2.5 mg/dL)
Coefficient of Variation (CV)	3.8%	2.8%

Intermediate Precision

Ten replicate assays drawn from 3 strip lots were run on each meter. These tests were run each day for a concentration levels were used in the testing. The following intermediate precision estimates were obtained.

Control Solution Level	MEAN
Low (CTRL 0)	1.9 mmol/L (33.5 mg/dL)
Normal (CTRL 1)	6.4 mmol/L (115.3 mg/dL)
High (CTRL 2)	18.8 mmol/L (338.9 mg/dL)

System Accuracy

The capillary blood glucose measurements from the On Call[®] Plus and On Call[®] EZ II Blood Glucose Monitoring Systems were obtained from fingertip, palm and forearm samples. Meter testing. Fingertip samples from the same person were tested on the 2300 STAT PLUS Glucose Analyzer (x). The results are shown below.

Linear Regression Results: On Call [®] Plus		
Sample Site	Slope	Intercept
Fingertip	0.9962	-0.0018
Palm	0.9954	-0.0018
Forearm	0.9930	-0.0018

On-Call[®] Plus

Blood Glucose Test Strips Age Insert

REF G133-111, REF G137	English
REF G133-118, REF G13	

PRINCIPLE INTENDED USE

The On Call[®] Plus Blood Glucose Test Strips are thin strips with a chemical reagent system. They work with the On Call[®] Plus or On Call[®] EZ II Blood Glucose Meters to measure the glucose concentration in whole blood. Blood is applied to the end tip of the test strip, then automatically absorbed into the reaction cell where the reaction takes place. A transient electrical current is formed during the reaction and the blood glucose concentration is calculated based on the electrical current detected by the meter, then the result is shown on the meter display. The meters are calibrated to display plasma-like concentration results. For *in vitro* diagnostic use. Test strips are to be used only outside the body for testing purposes. For self-testing and professional use. On Call[®] Plus Test Strips are used by people with diabetes at home and by healthcare professionals for the quantitative measurement of glucose in capillary whole blood from the finger, forearm, and palm.

COMPOSITION

Each test strip contains the following reactive chemicals: Glucose oxidase < 25 IU, Mediator < 300 µg. Each test strip vial contains a drying agent.

STORAGE AND HANDLING

- Test strips should be stored in their protective vial. The vial's cap must be tightly closed. This is to keep the test strips in good working condition.
- Store test strips in a cool, dry place at room temperature, 2-35°C (36-95°F). Store them away from heat and direct sunlight.
- Do not freeze or refrigerate.
- To ensure accurate results, use the test strips at room temperature.
- Do not store the test strips outside their protective vial. Test strips must be stored in the original vial with the cap tightly closed.
- Do not store or use the test strips in a humid place such as a bathroom.
- Do not store the meter, the test strips or control solution near bleach or cleaners that contain bleach.
- Do not transfer the test strips to a new vial or any other container.
- Replace the vial cap immediately after removing a test strip.
- Use the test strip immediately after removing it from the vial.
- Do not use your test strips past the unopened expiration date printed on the vial. Using test strips past the expiration date may produce incorrect test results.
- Note:** All expiration dates are printed in Year-Month-Day format.
- Use a new vial of test strips for only 6 months after opening. The opened vial expiration date is 6 months after the vial was first opened. Write the opened vial expiration date on the vial label after you open it.

PRECAUTIONS

- For *in vitro* diagnostic use. The test strips are to be used only outside the body for testing purposes.
- Do not use test strips after the expiration date shown on the vial. Expired test strips may give incorrect blood glucose readings.
- Do not use test strips that are torn, bent, or damaged in any way. Do not reuse test strips.
- The sample must only be applied to the tip of the test strip. Do not apply blood or control solution to the top of the test strip as this may result in an inaccurate reading.
- Before running a blood glucose test, make sure that the code chip contained with that vial of strips is inserted into the code chip slot on the right side of the meter.
- Discard the vial and any unused test strips 6 months after you first open it. Constant exposure to air may destroy chemicals in the test strip. This damage can cause incorrect readings.
- Keep the test strip vial away from children and animals.
- Consult your physician or healthcare professional before making any changes in your treatment plan based on your blood glucose test results.
- These products are labeled with the  symbol to distinguish them from earlier product versions that were subject to the EN ISO 15197:2015 non-compliance. Test strips with the symbol are compatible only with the control solutions that have the same symbol. The symbol can be found on the boxes, labels and individual pouches.

MATERIALS PROVIDED

- Test Strips
- Code Chip
- Package Insert

MATERIALS REQUIRED BUT NOT PROVIDED

- Meter
- Sterile Lancets
- Lancing Device
- Control Solution

INSTRUCTIONS FOR USE

See your User's Manual for complete instructions for blood sample collection before use.

- Open the cap of the test strip vial only to remove a test strip for testing. Replace the cap immediately to protect the remaining test strips from moisture in the air.
- Run the blood glucose test following the instructions contained in your User's Manual.
- The blood glucose test result will be shown on the meter display window. This result should fall within the target range recommended by your healthcare professional. If your blood glucose test results are higher or lower, ask your healthcare professional what to do. Always consult your healthcare professional before making any changes to your treatment plan.

IMPORTANT: On Call[®] Plus and On Call[®] EZ II Blood Glucose Monitoring Systems allow alternative site testing for forearm and palm testing in addition to fingertip testing. There are important differences between forearm, palm and fingertip samples that you should know. Important information about forearm and palm glucose testing:

- When blood levels are changing rapidly such as after a meal, insulin dose or exercise, blood from the fingertips may show these changes more rapidly than blood from other areas.
- Fingertips should be used if testing is within 2 hours of a meal, insulin dose or exercise and any time you feel glucose levels are changing rapidly.
- You should test with the fingertips anytime there is a concern for hypoglycemia or you suffer from hypoglycemia unawareness.

RANGE OF EXPECTED BLOOD GLUCOSE VALUES

Blood glucose monitoring requires the help of a person who knows their own range of expected blood glucose values and the meaning of your blood glucose results.

Expected blood glucose levels for people without diabetes:

Time	Range
Fasting and Before Meals	70 - 100 mg/dL
2 Hours After Meal	Less than 140 mg/dL

CHECKING THE METER

Your blood glucose meter must be handled according to the instructions for meter care. The quality control test strips are working together properly. Follow the instructions for a quality control test. Three ranges CTRL 0, CTRL 1 and CTRL 2. Control Solution 1 is sufficient for most meters. If your strips may not be working correctly, you may want to contact your dealer for information on purchasing control solution. For confirmation of results, Control Solution 0 test strips should fall within the CTRL 1 range. For CTRL 1 tests should fall within the CTRL 1 range. When testing with Control Solution 1, the results to the CTRL 1 range on the vial label. **CAUTION:** If your quality control test result falls outside the CTRL 1 range, DO NOT use the system to test your blood glucose. If you cannot correct the problem, contact your dealer.

LIMITATIONS

- The On Call[®] Plus and On Call[®] EZ II meters are not intended for use with On Call[®] Plus and On Call[®] EZ II Blood Glucose Monitoring Systems tested and proven to work together effectively.
- Use only with whole blood. Do not use with plasma or serum.
- Do not use for testing neonates.
- Very high (above 60%) and very low (below 20%) blood glucose levels may affect the accuracy of the results. Talk to your healthcare professional to find out more.
- Abnormally high levels of vitamin C and other substances may affect the accuracy of the results.
- The system is tested to accurately read the range of 1.1-33.3 mmol/L (20 - 600 mg/dL).
- Fatty substances (triglycerides) up to 3,000 mg/dL may affect the accuracy of the results.
- The On Call[®] Plus and On Call[®] EZ II Blood Glucose Monitoring Systems are shown to work properly up to 10,000% (300 mg/dL).
- Severely ill persons should not run the glucose monitoring system.
- Blood samples from patients in shock, or with severe hypoglycemia, or with hyperosmolar state (with or without ketosis) may give inaccurate results when tested with On Call[®] Plus and On Call[®] EZ II Blood Glucose Monitoring Systems.
- Dispose of blood samples and materials carefully. Do not dispose of materials in the trash.

PERFORMANCE CHARACTERISTICS

The On Call[®] Plus and On Call[®] EZ II Blood Glucose Monitoring Systems meet the requirements of EN ISO 15197:2015 (*in vitro* diagnostic systems for self-testing in blood glucose monitoring systems for self-testing in blood glucose monitoring systems) and On Call[®] EZ II meters are calibrated by the Analyzer reference instrument, which is traceable to the International System of Units (SI).

Reproducibility, Precision

Ten replicate assays were each run on ten On Call[®] Plus and On Call[®] EZ II meters. The following reproducibility, precision estimates were obtained:

MEAN	2.1 mmol/L (38 mg/dL)	4.9 mmol/L (89 mg/dL)
Standard Deviation mmol/L (mg/dL)	0.08 mmol/L (1.4 mg/dL)	0.14 mmol/L (2.5 mg/dL)
Coefficient of Variation (CV)	3.8%	2.8%

Intermediate Precision

Ten replicate assays drawn from 3 strip lots were run on each meter. These tests were run each day for a concentration levels were used in the testing. The following intermediate precision estimates were obtained:

Control Solution Level	MEAN
Low (CTRL 0)	1.9 mmol/L (33.5 mg/dL)
Normal (CTRL 1)	6.4 mmol/L (115.3 mg/dL)
High (CTRL 2)	18.8 mmol/L (338.9 mg/dL)

System Accuracy

The capillary blood glucose measurements from the On Call[®] Plus and On Call[®] EZ II Blood Glucose Monitoring Systems were obtained from fingertip, palm and forearm samples. Meter testing. Fingertip samples from the same person were tested on the 2300 STAT PLUS Glucose Analyzer (x). The results are shown below:

Linear Regression Results: On Call [®] Plus		
Sample Site	Slope	Intercept
Fingertip	0.9962	-0.001
Palm	0.9954	-0.001
Forearm	0.9930	-0.001