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EN

A HYPERTONIC SEA WATER SOLUTION CONTAINING SEA WATER FROM BRITTANY WITH ORGANIC ALOE VERA GEL, ORGANIC ORANGE ESSENTIAL OIL, COPPER GLUCONATE AND CARBOXYMETHYL BETAGLUCAN.

USE. The Otosan® Nasal Spray Baby medical device is a nasal decongestant for topical use. It includes the properties of a hypertonic sea water solution containing sea water from Brittany and the combined action of organic Aloe Vera Gel, organic Orange Essential Oil, Copper gluconate and Carboxymethyl Betaglucan. The multi-action of Otosan® Nasal Spray Baby acts locally by deeply cleansing the nasal cavities, thus freeing the nose from excess mucus and helping to reduce nasal congestion. Thanks to its delicate formula and the fact that it is free from vasoconstrictors, this product is particularly suitable for children, pregnant women or women that are breastfeeding, and people with heart disease; in these cases, please always consult your doctor before using the medical device. Free from propellant gas.

INDICATIONS. Otosan® Nasal Spray Baby is ideal as a first adjuvant therapy for inflammatory processes, such as different types of rhinitis (viral, bacterial or allergic), sinusitis and dryness of the nasal mucosa, and for post-surgery.

DOSAGE. 1 spray per nostril, 2-4 times a day for 10-15 days.

DESCRIPTION. Otosan® Nasal Spray Baby is a useful adjuvant for treating acute rhinitis or rhinosinusitis, regardless of the cause of its origin. The product components help to expel excess mucus and have a protective, softening and soothing effect on the irritated mucosa. At the same time, Otosan® Nasal Spray Baby cleanses nasal cavities thanks to the hypertonic solution and the action of several components, which extract water from the mucosa, thus fluidifying mucus and helping to remove viruses, bacteria, pollen and macro and micro particles. The medical device is also useful in the event of excessive dryness of the nasal mucosa. Otosan® Nasal Spray Baby provides relief and creates a hostile environment for bacterial growth, thus preventing relapse.

TOLERABILITY. The absence of vasoconstrictors and the gentle action of its active ingredients make Otosan® Nasal Spray Baby a well tolerated product. Until today, no side effects or contraindications are known,

OTOSAN®

ISOPHARM
OTOSAN
NASAL BABY

93.50 dhs

NASAL
SPRAY

Baby

OTOSAN®

NATURAL PRODUCTS FOR A BETTER LIFE

however they could cause rare cases of anosmia; in the event of adverse effects, discontinue treatment immediately and consult a doctor. The bottle is strictly for personal use; it might carry infections.

A MULTI-ACTION PRODUCT TO GENTLY CLEANSE NASAL CAVITIES WITH A QUICK DECONGESTANT EFFECT.

SEA WATER (CONCENTRATE)

= CLEANSING AND SOFTENING ACTION

The sea water contained in this formula comes from the sea near Brittany. Its composition is similar to a saline solution and it is particularly rich in mineral nutrients and trace elements that are essential for our body. In addition to cleansing the nasal cavities and removing dust and pollen, hypertonic sea water favours mucociliary clearance and fluidity of excess mucus by favouring its expulsion.

ORGANIC ALOE VERA GEL

= PROTECTIVE AND HYDRATING ACTION

The polysaccharides found in the Aloe Vera Gel interact with mucin, which is present on the surface of the nasal cavities, thus forming an adhesive gel that protects the mucosa from irritants and microorganisms. Moreover, this protection hydrates the irritated surface, thus reducing pain, inflammation and swelling.

ORGANIC ORANGE ESSENTIAL OIL

= RELAXING AND PURIFYING ACTION

Orange Essential Oil stands out for its fresh aroma of citrus fruits. Among its many properties, it protects against external agents, such as microbes and bacteria. It is also known for its relaxing properties, since it eases nervous tension and contrasts insomnia, which may arise in connection with common colds.

COPPER GLUCONATE

= PROTECTIVE ACTION FOR THE MUCOSA

Copper gluconate is a trace element that protects the nasal mucosa since it is a toxic substance for many pathogenic microorganisms, including flu viruses and adenoviruses.

CARBOXYMETHYL BETAGLUCAN

= FLUIDIFYING AND MUCO-ADHESIVE ACTION

It is a polysaccharide derived from betaglucan and obtained by extracting it from the cell walls of yeasts and fungi. It makes nasal mucus more fluid and favours mucociliary movement. Furthermore, since it is mucus-adhe-

ENT complications.

and in post-endonasal surgery: 1 to 4 sprays daily in each nostril, when determined by a doctor.

HYPERTONIC Decongestant

SYMPTÔMES:

Nasal congestion /
Blocked nose

INDICATIONS:

Colds, (+ Allergies,
Allergic rhinitis:
Hypertonic 135ml)

Baby

Baby Hypertonic ⁽¹⁾⁽²⁾

From 2 months
to 2 years old 60 ml

For nasal congestion due to cold, rhinitis and rhinopharyngitis:
2 to 3 times a day, a 1 to 2 seconds spray in each nostril.

Adults
&
Children

Hypertonic ⁽¹⁾⁽²⁾ 135 ml
From 2 years old

In case of allergic rhinitis, cold and sinusitis:
2 to 3 times a day, a 1 to 3 seconds spray in each nostril, up to 6 times a day.

Eucalyptus ⁽³⁾ 135 ml
From 6 years old

For the relief of nasal congestion:
2 to 3 times a day, a 1 second spray ONLY in each nostril.

PHYSIOMER® Nasal Aspirator and PHYSIOMER® Unidoses 20 x 5 ml or 30 x 5 ml, also available. **PHYSIOMER® Hypertonic and PHYSIOMER® Eucalyptus** Sprays are also available in convenient 20 ml pack. Products availability changes according to countries.

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PHYSIOMER® is a complete range of natural seawater nasal care products.

ISOTONIC

Hygiene,
Active prevention

SYMPTOMS:

Runny nose
Stuffy nose

INDICATIONS:

Colds - Daily Nasal
cleansing for hygiene and
prevention, Prevents

FOR WHO

Baby

PRODUCTS

Baby ⁽¹⁾⁽²⁾ 115 ml
From 15 days of age

**Adults
& Children**

Kids ⁽¹⁾⁽²⁾ 115 ml
From 2 years old

Normal Jet ⁽¹⁾⁽²⁾ 135 ml
From 6 years old

Strong Jet ⁽¹⁾⁽²⁾ 210 ml
From 10 years old

RECOMMENDED USE

In treatment of nasal symptoms, in case of cold or rhinopharyngitis: 4 to 6 times a day, spray 1 to 2 seconds in each nostril. **In daily use, in hygiene and prevention of nasal infections:** 1 to 2 times a day, spray 1 to 2 seconds in each nostril.

In treatment of nasal symptoms, in case of cold (runny and/or stuffy nose) or flu: 4 to 6 times a day, spray 1 to 2 seconds in each nostril. **In daily use, in hygiene and prevention:** 1 to 2 times a day, spray 1 to 2 seconds in each nostril.

In treatment of nasal symptoms, in case of stuffy nose: 4 to 6 times a day, spray 1 to 2 seconds in each nostril. **In daily use, in hygiene and prevention:** 1 to 2 times a day, spray 1 to 2 seconds in each nostril.

In case of scabs or thick, sticky, purulent nasal discharge: 1 to 4 sprays daily in each nostril. **In case of chronic nasal discomfort, rhinosinusal pathologies**

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