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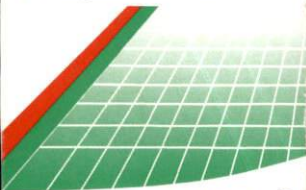
STÉRIMAR™

NOSE
HYGIENE
AND COMFORT

- Gently cleanses and moisturizes
- Helps to breathe better
- Prevents colds

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Sea water
microdiffusion
100% natural

Rich in
TRACE ELEMENTS
& MARINE MINERALS

PHYSIOLOGICAL
CHILDREN & ADULTS

Since 1975, STÉRIMAR™, pioneer of nasal hygiene, has put its expertise of the sea and its natural resources at the service of nasal health.

INDICATION OF USE:

STÉRIMAR™ Nose Hygiene and Comfort is a medical device recommended for children and adults to:

- Cleanse and eliminate impurities (dust, mucus, allergens...).
- Moisturize and restore the natural moisture of the nasal mucosa (heating, air conditioning).
- Prevent ENT disorders (colds, rhinitis, sinusitis, ear infections...) by reinforcing the nasal mucosa's defences.
- Help to improve breathing, contributing to a healthy sleeping pattern.

STÉRIMAR™ Nose Hygiene and Comfort may be used following nose surgery.

RECOMMENDED USE:

On first use, spray once to start the system.

For hygiene and moisturizing of the nasal mucosa: Head straight, put the nozzle in the nostril. Spray once or twice in each nostril. Let any excess solution flow out and then blow your nose.

For a deeper cleansing of nasal cavities: Head tilted to one side, put the nozzle in the upper part of the nostril. Spray once for longer, from 2 to 3 seconds. Let any excess solution flow out of the other nostril and then blow your nose.

Use 2 to 6 times a day.

After each use, clean the nozzle in hot water and dry.

COMPOSITION:

STÉRIMAR™ Nose Hygiene and Comfort is a physiological sea water solution* that contains a similar concentration of salt as that of the human body's cells.

Filtered, 100% natural and preservative-free, STÉRIMAR™ sea water benefits from the recognised properties of mineral salts and marine trace elements such as zinc, copper, manganese, iron, magnesium, selenium...

Sea water: 31.82 ml - Purified water qsp 100 ml. Aseptically packaged. Pressurised container with an inert gas (nitrogen). The solution is held within an inner container and does not come into contact with the nitrogen.

Net volume : 100 ml e 175
Approximately 300 sprays

Warning: Pressurised container. May burst if heated. Keep away from heat, hot surfaces, sparks, open flames and other ignition sources. No smoking. Do not pierce or burn, even after use. Protect from sunlight. Do not expose to temperatures exceeding 50°C/122 °F. Keep out of reach of children.

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PROMOPHARM S.A.