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Blood Glucose Test Strip

Features:

- User friendly design
- Only 1.4µL blood sample required
- Less blood means less pain
- Noble metal Electrode Strip performs high Precision and Accuracy

GS300**BIONIME**

Blood Glucose Test Strip

25 PCS (a vial of 25)**BIONIME**

Blood Glucose Test Strip

For *in vitro* diagnostic use

For self-testing and professional use

- Store between temperature 4°C and 30°C (39°F and 86°F)
- Do not freeze
- Read package insert before using
- Use within 3 months of opening strip vial
- Do not use if expiration date has passed
- Do not use if seals are broken
- Discard test strip properly after single use

Noble Metal Electrode Strip
(Smart Code Key included)**Rightest™ GS300****Use with Rightest™ GM300**101-2GS300-1G2
EN**BIONIME**

Blood Glucose Test Strip

Instruction of use GS300

1. Insert the strip into the socket on the top of the meter and wait for the information on the blood glucose meter LCD to appear.
2. After obtaining a blood sample on your finger, gently touch the tip of the strip in order for the blood sample to be absorbed.

**Rightest™ GS300****BIONIME CORPORATION**

No. 100, Sec. 2, Daqing St., South Dist.,
Taichung City 40242, Taiwan
Tel: +886 4 23692388
Fax: +886 4 22617586
http://www.bionime.com
E-mail: info@bionime.com

CE REP

BIONIME GmbH

Tramstrasse 16, 9442 Berneck,
Switzerland
E-mail: info@bionime.ch

BIONIME

Blood Glucose Test Strip

2023-02-03

LOT

212122213

**BIONIME**

Blood Glucose Test Strip

Features:

- User friendly design
- Only 1.4µL blood sample required
- Less blood means less pain
- Noble metal Electrode Strip performs high Precision and Accuracy

GS300**BIONIME**

Blood Glucose Test Strip

25 PCS (a vial of 25)**BIONIME**

Blood Glucose Test Strip

For *in vitro* diagnostic use

For self-testing and professional use

- Store between temperature 4°C and 30°C (39°F and 86°F)
- Do not freeze
- Read package insert before using
- Use within 3 months of opening strip vial
- Do not use if expiration date has passed
- Do not use if seals are broken
- Discard test strip properly after single use

Noble Metal Electrode Strip
(Smart Code Key included)**Rightest™ GS300****Use with Rightest™ GM300**101-2GS300-1G2
EN**BIONIME**

Blood Glucose Test Strip

Instruction of use GS300

1. Insert the strip into the socket on the top of the meter and wait for the information on the blood glucose meter LCD to appear.
2. After obtaining a blood sample on your finger, gently touch the tip of the strip in order for the blood sample to be absorbed.

**Rightest™ GS300****BIONIME CORPORATION**

No. 100, Sec. 2, Daqing St., South Dist.,
Taichung City 40242, Taiwan
Tel: +886 4 23692388
Fax: +886 4 22617586
http://www.bionime.com
E-mail: info@bionime.com

CE REP

BIONIME GmbH

Tramstrasse 16, 9442 Berneck,
Switzerland
E-mail: info@bionime.ch

BIONIME

Blood Glucose Test Strip

2023-02-03

LOT

212122213

**BIONIME**

Blood Glucose Test Strip

Features:

- User friendly design
- Only 1.4µL blood sample required
- Less blood means less pain
- Noble metal Electrode Strip performs high Precision and Accuracy

GS300**BIONIME**

Blood Glucose Test Strip

25 PCS (a vial of 25)**BIONIME**

Blood Glucose Test Strip

For *in vitro* diagnostic use

For self-testing and professional use

- Store between temperature 4°C and 30°C (39°F and 86°F)
- Do not freeze
- Read package insert before using
- Use within 3 months of opening strip vial
- Do not use if expiration date has passed
- Do not use if seals are broken
- Discard test strip properly after single use

Noble Metal Electrode Strip
(Smart Code Key included)**Rightest™ GS300****Use with Rightest™ GM300**101-2GS300-1G2
EN**BIONIME**

Blood Glucose Test Strip

Instruction of use GS300

1. Insert the strip into the socket on the top of the meter and wait for the information on the blood glucose meter LCD to appear.
2. After obtaining a blood sample on your finger, gently touch the tip of the strip in order for the blood sample to be absorbed.

**Rightest™ GS300****BIONIME CORPORATION**

No. 100, Sec. 2, Daqing St., South Dist.,
Taichung City 40242, Taiwan
Tel: +886 4 23692388
Fax: +886 4 22617586
http://www.bionime.com
E-mail: info@bionime.com

CE REP

BIONIME GmbH

Tramstrasse 16, 9442 Berneck,
Switzerland
E-mail: info@bionime.ch

BIONIME

Blood Glucose Test Strip

2023-02-03

LOT

212122213

**BIONIME**

Blood Glucose Test Strip

Features:

- User friendly design
- Only 1.4µL blood sample required
- Less blood means less pain
- Noble metal Electrode Strip performs high Precision and Accuracy

GS300**BIONIME**

Blood Glucose Test Strip

25 PCS (a vial of 25)**BIONIME**

Blood Glucose Test Strip

For *in vitro* diagnostic use

For self-testing and professional use

- Store between temperature 4°C and 30°C (39°F and 86°F)
- Do not freeze
- Read package insert before using
- Use within 3 months of opening strip vial
- Do not use if expiration date has passed
- Do not use if seals are broken
- Discard test strip properly after single use

Noble Metal Electrode Strip
(Smart Code Key included)**Rightest™ GS300****Use with Rightest™ GM300**101-2GS300-1G2
EN**BIONIME**

Blood Glucose Test Strip

Instruction of use GS300

1. Insert the strip into the socket on the top of the meter and wait for the information on the blood glucose meter LCD to appear.
2. After obtaining a blood sample on your finger, gently touch the tip of the strip in order for the blood sample to be absorbed.

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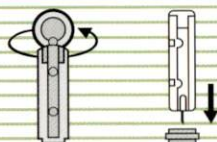
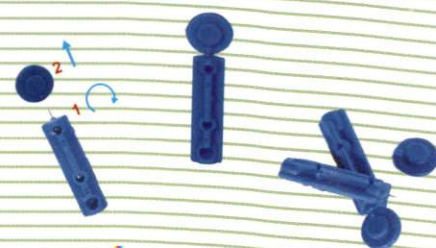
Manufactured By:
Tianjin Huahong Technology Co., Ltd.
A01, Plant B, No.278, Hangkong Road,
Tianjin Pilot Free Trade Zone (Air Port
Industrial Park), Tianjin 300308, China
Distributed By:
Astute SARL
Unite Spl # 3, Hail Mohamadie #15
Marrakech, Morocco

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GM300

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Our value :

Offering a worry-free testing experience, with lasting reliable results, professional performance, to be your everyday health partner.

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BIONIME

BIONIME CORPORATION

No.100, Sec. 2, Daqing St.,
South Dist., Taichung City
40242, Taiwan

P. +886 4 2369 2388

F. +886 4 2261 7586

info@bionime.com

bionime.com

BIONIME

RIGHTEST GM300 BLOOD GLUCOSE MONITORING SYSTEM

PART NO.: 99GM300021

SERIAL NO.: 2300UAM3439

STRIP EXP DATE : 2022-11-25

STRIP LOT NO.: 2120C2509

Manufacturer: BIONIME CORP.

No.100, Sec. 2, Daqing St., South Dist.,
Taichung City 40242, Taiwan

4 710949 330422

IF PACKAGE CONTAINS STRIPS OR CONTROL SOLUTION,
PLEASE STORE AT TEMPERATURES UNDER 30°C (86°F). DO NOT FREEZE.

Lancet Manufacturer: SteriLance Medical (SuZhou) Inc.
No.68 Litanghe Road, Xiarfengcheng, Suzhou, 215133, China