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SD CodeFree™

BLOOD GLUCOSE MONITORING SYSTEM

SD CodeFree™ Blood Glucose Monitoring System

CONTENTS
Blood Glucose Meter (01GM11)
10 Blood Glucose Test Strips (01GS11A) (option)
1 STANDARD™ Glucose Check Strip (01GCH10)
1 STANDARD™ Glucose Control Solution - Level M (01GCS10-M) (option)
10 Lancets
Lancing device
3V battery type CR2032
Carrying case
User Instruction Guide
Self Test Diary

For in vitro diagnostic use only

SD CodeFree™ Blood Glucose Monitoring System is an in vitro
diagnostic medical device for self-testing.

Manufactured by
SD BIOSENSOR

Head office
C-4th&5th, 16, Deogyong-daero 1556beon-gil,
Yeonjong-gu, Suwon-si, Gyeonggi-do, 16690,
REPUBLIC OF KOREA
Manufacturing site
74, Osongsaengmyeong 4-ro, Osong-eup,
Heungdeok-gu, Cheongju-si, Chungcheongbuk-do,
28161, REPUBLIC OF KOREA

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Authorized Representative
MT Promed Consulting GmbH
Altenhofstrasse 80 D-66386 St. Ingbert Germany
Phone: +49 6894 581020, Fax: +49 6894 581021

Lancet
Refer to the separate label.
Lancing Device
Beijing Ruicheng Medical Supplies Co., Ltd.

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Σ 25 X 2 TEST

Manufactured by
SD Biosensor, Inc.

Head office
C-44h&5th, 16, Deogyeong-daem 1550beon-gil,
Yeongtong-gu, Suwon-si, Gyeonggi-do, 16690,
REPUBLIC OF KOREA.
Manufacturing site
74, Gwangju-gu 4-ro, Gwang-yeop,
Heungdeok-gu, Cheongju-si, Chungcheongbuk-do,
28161, REPUBLIC OF KOREA
www.sdbiosensor.com

Authorized Representative
MT Promed Consulting GmbH
Altenhofstrasse 80 D-46386 St. Ingbert Germany
Phone: +49 6894 581020, Fax: +49 6894 581021

MB03CF1MLR5
Issue date: 2018.05

NEW


OPEN
HERE

SD BLOOD GLUCOSE MONITORING SYSTEM
CodeFreeTM

Blood Glucose Test Strip

SD BLOOD GLUCOSE MONITORING SYSTEM
CodeFreeTM

Blood Glucose Test Strip **GOLD ELECTRODE**

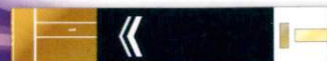SD BIOSENSOR

SD Biosensor is not related to Alere Inc. or to Standard Diagnostics, Inc.

LOT NO.: 80319054
MFG DATE: 2019. 03. 28
EXP DATE: 2021. 03. 27

INSTRUCTIONS FOR USE:

1.
Twist
Cap

2.
Pull
Cap

1. Wash hands with soap and warm water. Dry thoroughly.
2. Select fingertip site, slightly off-center.
3. Remove protective lancet cover; above (steps 1 & 2).
If seal is already broken discard lancet.
4. Insert lancet into lancing device, use to puncture skin.
5. Apply drop of blood to test strip, wipe excess from finger.
6. Recap lancet, discard in appropriate container.

Do not use if safety seal has been broken.

CE 0123

STERILE R

Twist Lancet

- Universal design fits almost all lancing device
- Single use only

200 Sterile Lancets

30G

Shanghai Carelife International Trading Co.,Ltd.
1707 Yinqiao Bldg.,58 Jinxin Rd. 201206 Shanghai, CHINA
Tel:0086 21 58542656

Shanghai International Holding Corp.GmbH(Europe)
Eiffestraße 80,20537 Hamburg,GERMANY

180128T

2023-01-27

6 951568 002712 >

Made in China